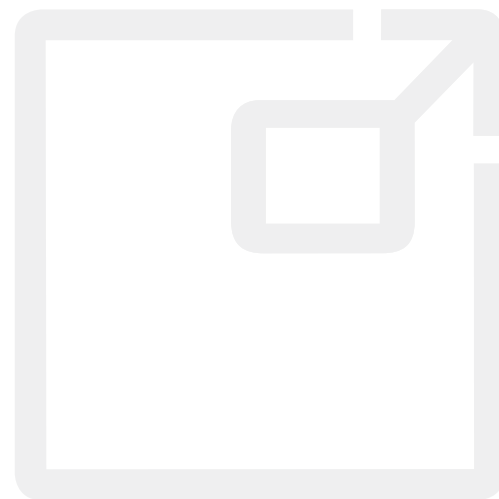
PHALLOSAN[®] forte

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REF 5000

INSTRUCTIONS FOR USE

Congratulations on purchasing your PHALLOSAN® forte! PHALLOSAN® forte has been used around the world with much success for 18 years.

Purpose and indications:

- PHALLOSAN® forte is recommended as a treatment for
- » penis enlargement-penis extension
 - » congenital/acquired penile curvature and induratio penis plastica (IPP)
 - » erectile dysfunctions
 - » post-surgical follow-up treatment after prostate surgery
 - » training the cavernous body of the penis at any age

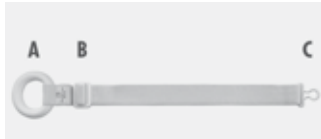
Package contents:

1. Elastic belt, which is strapped around the waist incl. foam-rubber ring (A), into which the penis is inserted all the way to the base. The buckle (B), with which you can adjust the length of the belt individually and adapter (C) on the belt, which is connected to the suction bell.
 2. a) Suction bell in size S with PHALLOSAN® forte – S-sleeve condom
 2. b) Suction bell in size M with PHALLOSAN® forte – M-sleeve condom
 2. c) Suction bell in size L with PHALLOSAN® forte – L-sleeve condom and assembled tension clip (3)
 3. Tension clip with tension spring attachment to adjust the intensity of the tension using a traffic light colour code system with adapter for the belt.
 - 4./5. Protector cap to protect glans against strong tension settings.
 6. Suction ball with 3-way valve
 7. Measuring template
- Instruction Guide
Traveller Bag with 4 extra sleeve condoms (1xS, 2xM, 1xL)

Preparation:


1. Belt

Spread the orthopaedic PHALLOSAN® forte belt out in front of you and adjust it to its longest length using the buckle (B).

2. Selection of the suction bell/sleeve

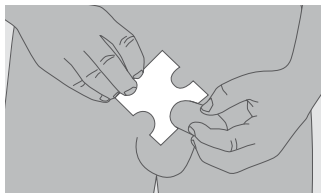Determine your correct suction bell size using the enclosed measuring template (7). Pull the flaccid penis away from your body and take measurement on the penis shaft.

The bell in size M (2c) is already connected to the sleeve size M (2c) and the tension clip with tension spring attachment in the set.

Should you require a different size, please remove the tension clip (3).

Now, please take your appropriate bell and adapt it using the suction ball; it only snaps into the position with the arrow symbol (↕). The appropriate sleeve (2) is slipped over the bead of the bell and the tension clip (3) is reattached.

When attaching to the bell, it must be observed that the bead of the bell snaps into the groove on the sleeve on the thick end of the sleeve. If the sleeve is pulled too far over the bead of the bell, the sleeve can rupture.

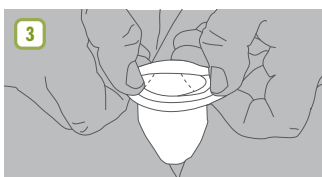
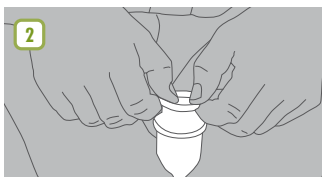
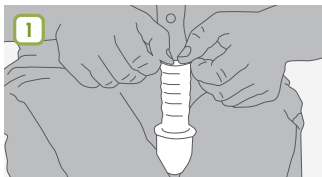
Top: Wrong, pulled too far over the bell and not powdered!

Bottom: Right and powdered.

Please observe that the sleeve loses its coating after several days. Therefore, you should purchase baby powder and apply this to the outer surface of the sleeve. It forms a protective coat, which protects the sleeve against friction, adhesion and tearing.

Clamp it the suction bell between your knees. With a slight pull, carefully roll the sleeve condom 0.40 inch to the edge of the suction bell and pull it behind the bead of the suction bell. Please be careful and do not damage the sleeve with sharp fingernails or the like. Do not pull hard when rolling back.

Special features of the new PHALLOSAN® forte sleeves

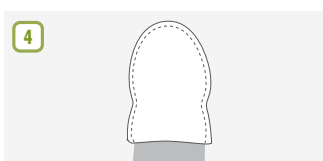
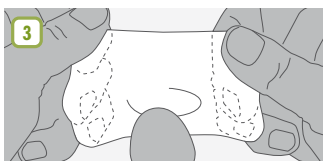
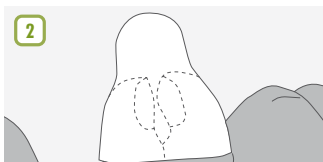
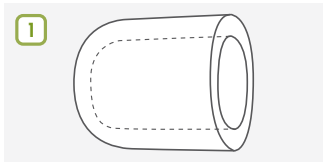
The PHALLOSAN® forte sleeves consist of medical silicone, thus ruling out the risk of irritation or

allergic reactions. This innovation not only offers you more comfort but also provides a uniquely snug fit around the penis. A part of it is approx. 3 mm thick, that is where it is attached to the bell; its thinnest area is 0.3 mm thick, to smoothly adapt to the shaft of your penis.

Your perfect suction bell is now ready for use!

3. Protector cap

Please remove the protector cap (4/5), which fits any penis size. Pull the protector cap apart using both hands and slip it over the glans. It should fit snugly.

After fitting the cap, it is important to make sure that as little air as possible remains at the tip of the glans. It can be helpful to slip the protector cap over the penis when the latter is in a slightly erected state. The protector cap should protect the glans against the effects of possible excess tension.

Experienced users can usually do away with the protector cap after a few weeks.

All necessary preparations for the application have now been completed.

Application:

1. Now, insert the penis including the protector cap into the suction bell and roll the sleeve back from the bead of the bell over the penis.

2. Turn the green protruding marker on the valve of the suction ball (6) to the suction symbol and press the suction ball several times with your fingers. By doing so, the glans is drawn further into the bell. The sleeve smoothly adapts to the shaft of the penis.

3. Now, turn the green protruding marker on the valve of the suction ball (6) to the lock symbol, to maintain the micro-vacuum.

4. Now, please take the belt previously spread out before you and slip the white foam-rubber ring (A) over the complete suction bell apparatus until it rests against the root of the penis.

5. Now, pull the end of the belt around your waist.

6. Turn the open end of the tension clip (3) towards your body and connect both eyelets (C).

7. Tighten the fit of the belt using the belt buckle (B) until you can feel that your penis is being stretched and the traffic light colour code system indicates green. Please make sure that the belt slants upwards in the groin area. To do this, pull the belt upwards around

the buttocks towards the waist. If the belt slips to the fold of the buttocks, i.e. parallel to the penis, the tension will be further increased when sitting, which may be unpleasant.

8. You can now carry on with your everyday business or also wear PHALLOSAN® forte while sleeping, given that your blood circulation is not restricted at any time.

Ending the application:

First, please take the belt off and turn the green valve of the suction ball to the arrow symbol. The vacuum escapes. Please roll the sleeve carefully back towards the hard plastic cup and remove the complete suction device. **Do not stretch the sleeve with your fingers, to remove the vacuum, as it will otherwise rip immediately!** It will be easily removed when simply rolling the sleeve upwards.

Special technical features:

Put simply, this apparatus is a suction bell with a seal. Therefore, it is not important how far the sleeve reaches over the penis shaft. The sleeve is not designed to transmit force to the penis. This solely takes place by the micro-vacuum formed in the bell.

Should you feel that the vacuum has slightly decreased while wearing the apparatus and the glans has possibly slipped too far out of the bell, this can be the result of several factors. In any case, air is penetrating. This can take place via the penis shaft if pubic hair was not diligently removed or a sleeve, which is too wide, was selected.

As far as necessary, please remove possible pubic hair from the penis. This can be done by shaving or using a mild hair removal cream.

Remedy

Turn the green protruding marker on the valve of the suction ball away from the lock symbol to the suction symbol. In this manner, draw the glans back into the bell and close the valve by

turning the green protruding marker back towards the lock symbol.

If the glans is still slipping out of the bell, the cause is definitely the sleeve, which is apparently too wide.

You have the option of adjusting this individually to your size and in doing so, achieving optimum efficiency. For example, if you feel that the M bell is the correct size but the sleeve is too large for the penis shaft, you can place the S sleeve on the M bell. You can do the same with the L bell and the M sleeve.

Under no circumstances can you place the S sleeve on the L bell. This can cause the material to rupture.

Instructions for care FORTE/PLUS:

The sleeve can and should be detached from the bell and should be washed in lukewarm water for hygienic reasons.

Keep the sleeve rolled up until you need it again.

Please never forget to apply baby powder to the outside of the sleeve after use when rolled out.

The suction bell should also be cleaned daily as described above. Do not forget to remove the suction ball first by moving the green protruding marker on the valve of the suction ball towards the double arrow. Carefully pull the suction ball out of the suction bell. The suction ball may not come in contact with water or oil because the sensitive valves would then no longer function efficiently.

Step-by-step plan for the precise adjustment of the tension of the belt to the connective tissue of the penis:

Preamble:

The body's connective tissue depends on the respective hereditary disposition. Therefore, we developed a traffic light colour code system, to enable you to adjust the belt tension precisely to your type of connective tissue. In any case,

please make sure that the tension is in the lower green range in the beginning. This will help you achieve optimum results for penis enlargement in length and width without any negative side-effects.

Day 1

Only set the tension in the lower green range and do not wear PHALLOSAN® for more than 1 hour. Remove the belt and turn the green protruding marker on the valve of the suction ball towards the arrow symbol. The vacuum escapes.

Carefully roll the sleeve back towards the glans and remove the suction bell apparatus.

Now, examine the point on the penis between the opening of the urethra and the frenulum on the underside of the glans. This is the most sensitive tissue on the penis. If it is reddened or swollen, the belt tension was too tight. Suspend the application for today. If you detect changes on the penis, do not begin application again until these have subsided.

Day 2 to 8

If you were able to wear PHALLOSAN® forte on the previous day without suffering from discolouration or swelling on the points in question, you can wear PHALLOSAN® one hour longer.

Extend the period of wear every day by one hour. After 8 days you will have adjusted to wearing the apparatus.

After the adjustment period PHALLOSAN® can also be worn at night. You will notice that you can feel nubs on both symbols for pumping and locking, which enables pumping or locking without having visual contact. The sleeve expands during night-time erections; circulation is not impeded, it is improved.

You will wear PHALLOSAN® forte with enthusiasm. Your penis can become measurably longer and thicker in only a few weeks. You will experience positive side effects such as a stronger sex drive and greater pleasure.

Period of application:

PHALLOSAN® should be worn 8–10 hours per day, 6 days a week, and at any time even longer!

Side effects/Precautions:

PHALLOSAN® forte is used at your own risk however, it does not pose any risks if used according to the instructions. Particularly during the first weeks of application, excess tension on the penis should be avoided.

If swelling and/or blisters occur on the foreskin or on the glans, the belt will have been set too tight. It is imperative to discontinue use until the swelling or blisters have disappeared. Then set the tension somewhat looser.

Occasionally you can feel pressure pain on the penis shaft; this can result from the bulging end of the sleeve resting on the silicone rubber-foam ring and therefore, cause pain.

All serious incidents relating to the device must be notified to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Remedy:

Slightly roll the end of the sleeve back from the silicone rubber-foam ring; pressure pain will no longer occur.

You can also distribute the tension of the belt over the body, to reduce tension on the root of the penis. Take hold of the belt where the suction bell is connected to it and lead the tension around the body to relieve the root of the penis. It sounds complicated but it is not; the tension is now distributed throughout the entire area of the belt.

Contraindications:

After consuming alcohol, after taking drugs as well as painkillers, sleeping pills and tranquilizers because your natural perception of pain may be compromised and you may not be able to promptly react to pain. Avoid using PHALLOSAN® forte in the event of infections or open wounds on the penis.

How do I measure the exact increase in the length of my penis?

Please use a foot rule. A measuring tape is not suitable. Hold your penis in a flaccid state away from your body (90 degrees). Position the foot rule on the side of your penis (as seen looking

down). Press the rule against your pubic bone.

Important:

Before beginning with the application, determine your exact body weight without clothing. In the event of a possible weight gain, a layer of fat forms — in particular — in the pubic area of the pubic hair around the penis. This layer of fat reduces the distance between the tip of the penis and the pubic bone, thus falsifying (shortening) the result. Physicians call this an abdominal apron.

Repeat orders:

All parts for PHALLOSAN® forte can be reordered individually from us. Should you have any other questions or suggestions, feel free to contact our experts at any time free of charge. You merely have to pay the normal telephone charges. At all times, we reserve the right to make technical modifications, which we will also make available on the Internet at

www.phallosan.com
www.phallosan.co.uk

Hotline:

Phone +423 235 84 77
(Liechtenstein, Mo–Fr 9:30–12:00,
14:00–16:00 TIME MEZ)
service@swissana.com

For more Information:
www.phallosan.info

Your PHALLOSAN® forte Team

INSTRUCTIONS FOR USE PHALLOSAN® VERTICAL

Please read the instructions for use before using PHALLOSAN® vertical. This is an extension set for PHALLOSAN® forte and is only to be used together with the PHALLOSAN® forte package!

Please visit www.phallosan.info for instructions on PHALLOSAN® forte!

Package contents PHALLOSAN® vertical:

- A. Knee cuff
- B. Elastic strap that goes around the knee cuff

Package contents PHALLOSAN® forte:

- 2. Hard plastic bell with attached sleeve condom
- 3. Tension clip with tension spring attachment to adjust the intensity of the tension using a traffic light colour code system with adapter for the belt
- 6. Integrated vacuum pump

Preparation:

Selection of the suction bell/sleeve

Determine your correct suction bell size using the enclosed measuring template (7). Pull the flaccid penis away from your body and take measurement on the penis shaft.

The bell in size M (2c) is already connected to the sleeve size M (2c) and the tension clip with

tension spring attachment in the set.

Should you require a different size, please remove the tension clip (3).

Now, please take your appropriate bell and adapt it using the suction ball; it only snaps into the position with the arrow symbol (↕). The appropriate sleeve (2) is slipped over the bead of the bell and the tension clip (3) is reattached.

When attaching to the bell, it must be observed that the bead of the bell snaps into the groove on the sleeve on the thick end of the sleeve. If the sleeve is pulled too far over the bead of the bell, the sleeve can rupture.

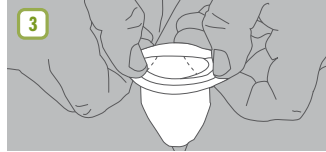
Top: Wrong, pulled too far over the bell and not powdered!

Bottom: Right and powdered.

Please observe that the sleeve loses its coating after several days. Therefore, you should purchase baby powder and apply this to the outer surface of the sleeve. It forms a protective coat, which protects the sleeve against friction, adhesion and tearing.

Clamp it the suction bell between your knees. With a slight pull, carefully roll the sleeve condom 0.40 inch to the edge of the suction bell and pull it behind the bead of the suction bell. Please be careful and do not damage the sleeve with sharp fingernails or the like. Do not pull hard when rolling back.





Special features of the new PHALLOSAN® forte sleeves

The PHALLOSAN® forte sleeves consist of medical silicone, thus ruling out the risk of irritation or allergic reactions. This innovation not only offers you more comfort but also provides a uniquely snug fit around the penis. A part of it is approx. 3 mm thick, that is where it is attached to the bell; its thinnest area is 0.3 mm thick, to smoothly adapt to the shaft of your penis.

Your perfect suction bell is now ready for use!

Protector cap

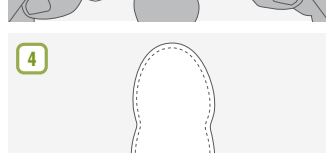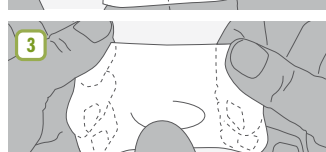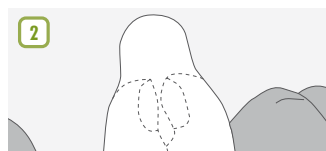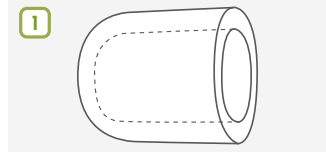Please remove the protector cap (4/5), which fits any penis size. Pull the protector cap apart using both hands and slip it over the glans. It should fit snugly.

After fitting the cap, it is important to make sure that as little air as possible remains at the tip of the glans. It can be helpful to slip the protector cap over the penis when the latter is in a slightly erected state. The protector cap should protect the glans against the effects of possible excess tension.

Experienced users can usually do away with the protector cap after a few weeks.

All necessary preparations for the application have now been completed.

Application of PHALLOSAN® forte:

- Now, insert the penis including the protector cap into the suction bell and roll the sleeve back from the bead of the bell over the penis.

- Turn the green protruding marker on the valve of the suction ball (6) to the suction symbol and press the suction ball several times with your fingers. By doing so, the glans is drawn further into the bell. The sleeve smoothly adapts to the shaft of the penis.

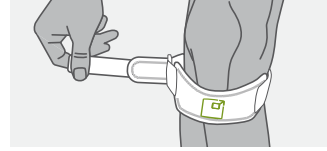
- Now, turn the green protruding marker on the valve of the suction ball (6) to the lock symbol, to maintain the micro-vacuum.

Application of PHALLOSAN® vertical:


- Open the Velcro fastener on the knee cuff (A). Place the cuff underneath one knee below the knee (right or left). Close the Velcro fastener. Make sure that the PHALLOSAN® logo is facing forward.

- Attach the belt: Take the shortened PHALLOSAN® vertical strap (B). Attach it to the inside of the knee cuff, the logo should be visible on the outside.

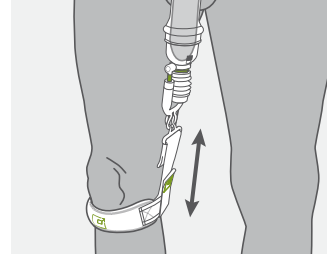- Adjust the pulling force: Attach the adapter to the tension clip. Adjust the pull strength using the long Velcro fastener on the strap without reaching the red area of the tension Clip.

- Wearing in everyday life: PHALLOSAN® vertical can now be worn comfortably under trousers.

To end the application and for further application instructions:

See PHALLOSAN® forte manual.

Contact:

E-mail: service@phallosan.com

More information at:
www.phallosan.info

Your PHALLOSAN® Team

INSTRUCTIONS FOR USE PHALLOSAN® PLUS+

Please read the instructions for use before using PHALLOSAN® plus+.

This is an extension set for PHALLOSAN® forte and is only to be used together with the PHALLOSAN® forte package!

Please visit www.phallosan.info for instructions on PHALLOSAN® forte!

Package contents PHALLOSAN® plus+:

1. C-Ring adapter for the connection to the suction cup
 - » suitable for any bell size
 - » flexible movement
 - » safety clip
 - » easy removal
2. The rod system with white base ring
 - » extendable to any penis size
 - » optimal tension
 - » option of high or comfortable strength
3. Long tube
 - » for penis lengths from 12–22 cm / 4.7–8.7 inches

Package contents PHALLOSAN® forte:

4. Hard plastic bell with attached sleeve condom
 - » up to 4 sizes for each member type
 - » transparent safety
 - » optional intermediate size (large bell, medium sleeve)
 - » anti-allergic material
 - » 3-valve system
 - » comfort combines with effectiveness
5. Integrated vacuum pump
 - » generates a micro-vacuum
 - » no damage to tissue/erectile bodies/veins
 - » improved result
 - » maximum performance

Preparation

Attaching the suction cup and protector cap

Please observe and follow the instructions provided in PHALLOSAN® forte „2 instructions for use. Selection of the suction cup up to paragraph 3 in the instructions for use

1. Preparation of PHALLOSAN® plus+ rod device:

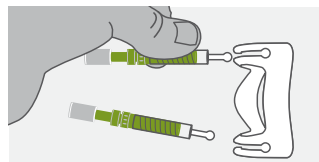
Use a meter ruler to determine which of the two transparent rod sizes you need for PHALLOSAN® plus+. Using one hand, pull the penis lengthwise behind the head of the penis. Place the meter ruler along the side of the penis with the other hand so that the measurements are easier to read.

MEDIUM – Penis length 8–17 cm / 3–6.7 inches, is already assembled.


LARGE – Penis length 12–22 cm / 4.7–8.7 inches, transparent threaded tubes, position 3 in the packaging.

2. Conversion from medium to large (3):


- a. Before doing so, pull off the upper adapter device together with the clip. Remove the shorter tubes by pulling them diagonally forwards, out of the thread on the white base ring. Remove the two longer tubes from the packaging (3).

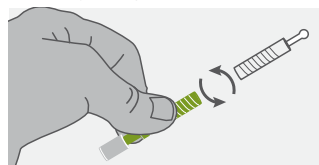- b. Now unscrew the thicker green tubes from the shorter transparent tubes and mount this on the longer transparent threaded tubes.

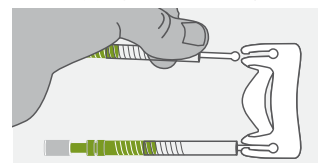
- c. Now press the tubes onto the ball joint socket of the white base ring (2). Try to angle the tubes onto the joint and press them in, this makes the engagement with the head of the ball joint easier. Now reassemble the adapter device (1) together with the C-clip.

3. Instructions for use PHALLOSAN® plus+:

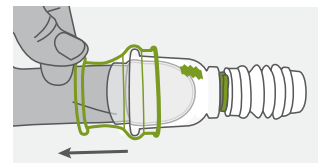- a. Hold PHALLOSAN® plus+ and insert the penis through the white base ring. It is easiest to use in a sitting position. PHALLOSAN® plus+ now lies loosely against the penis shaft with the white base ring resting against the thigh. Now take the ready-to-use hard plastic bell/with adapted (rolled up) sleeve and pump.


Place the head of the penis as far as possible in the bell, operate the pump 3-5 times until the head of the penis is well inside the bell and turn the green valve to the lock symbol.


Now unroll the sleeve condom over the shaft of the penis.

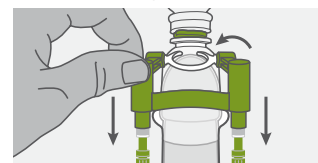- b. Finally, push down the rods with the adapter device, attach the C-clip to the hard plastic bell with the opening facing the stomach and then release. The penis is now under tension.

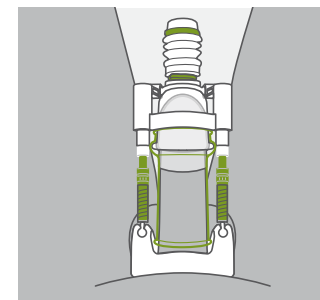- c. You are now able to adjust the pulling power individually with the unique thread. Please ensure even adjustment. You should ensure that no discomfort is felt by the penis at any time.

4. Removal of the PHALLOSAN® plus+:

First remove the C-clip from the hard plastic bell.

Now turn the green valve of the suction cup to the arrow symbol. The vacuum is released. Please roll the sleeve carefully back towards the hard plastic bell to remove the entire sleeve condom along with the suction bell. Avoid stretching the sleeve with your fingers as this could cause it to tear. It is removed without difficulty by simply rolling the sleeve back from the shaft of the penis.

5. Side effects/precautionary measures:

PHALLOSAN® plus+ is used at the user's own risk, but is safe if the instructions for use are followed. Nevertheless, strong pulling power should not be exerted on the penis, especially during the first few weeks of use. Should swelling and/or blisters appear on the foreskin or head of the penis, the setting is too strong. Be sure to take a break until the swelling or blisters have healed.

Adjust the pull to a slightly lighter setting afterwards. Please be patient, the procedure is comparable in its application success with an adjustment of the jaw. The continuity of stretching, not just the strength of the pull, is the guarantor of success!

Please note: The use of PHALLOSAN® plus+ is not suitable for use while you are sleeping!

Now you have the perfect combination, whatever the purpose:

Penis enlargement:

Use PHALLOSAN® plus+ at least 6 hours a day, 5 days a week. This is not about a high pulling power, achieving cell stimulation depends on continuity.

As a result, new cells are stimulated and an increase in length of 4.9 cm (1.9 inches) and circumference of 2.5 cm (1.1 inches) is possible.

Erectile dysfunction:

5 hours 3 days a week leads to greatly improved blood flow, stimulating your prostate and sexual desire.

The erectile tissue is then able to absorb more blood and erections are more frequent and longer lasting.

IPP (Induratio penis Plastica), curvature of the penis:

IPP/penile curvature is possible to treat with a combination set of PHALLOSAN® forte and PHALLOSAN® plus+.

Important: Combine PHALLOSAN® forte alternately with PHALLOSAN® plus+. The continuity of stretching creates painless microfibre tears in the plaque tissue of the curvature. This is where new cell tissue is embedded.

In doing so, it is possible to straighten the penis.

Contact:

E-mail: service@phallosan.com

More information at:
www.phallosan.info

Your PHALLOSAN® Team

The PHALLOSAN forte app. Your personal training companion.

With the **free PHALLOSAN® forte app** you can optimise the achievement of your goals and increase your motivation.

The app reminds you when it is time to wear PHALLOSAN® forte, because regular training is crucial for success. You receive uncomplicated statistics on the daily wearing time and the progress of your training. The intuitive menu navigation and visualisation also motivate you to keep going.

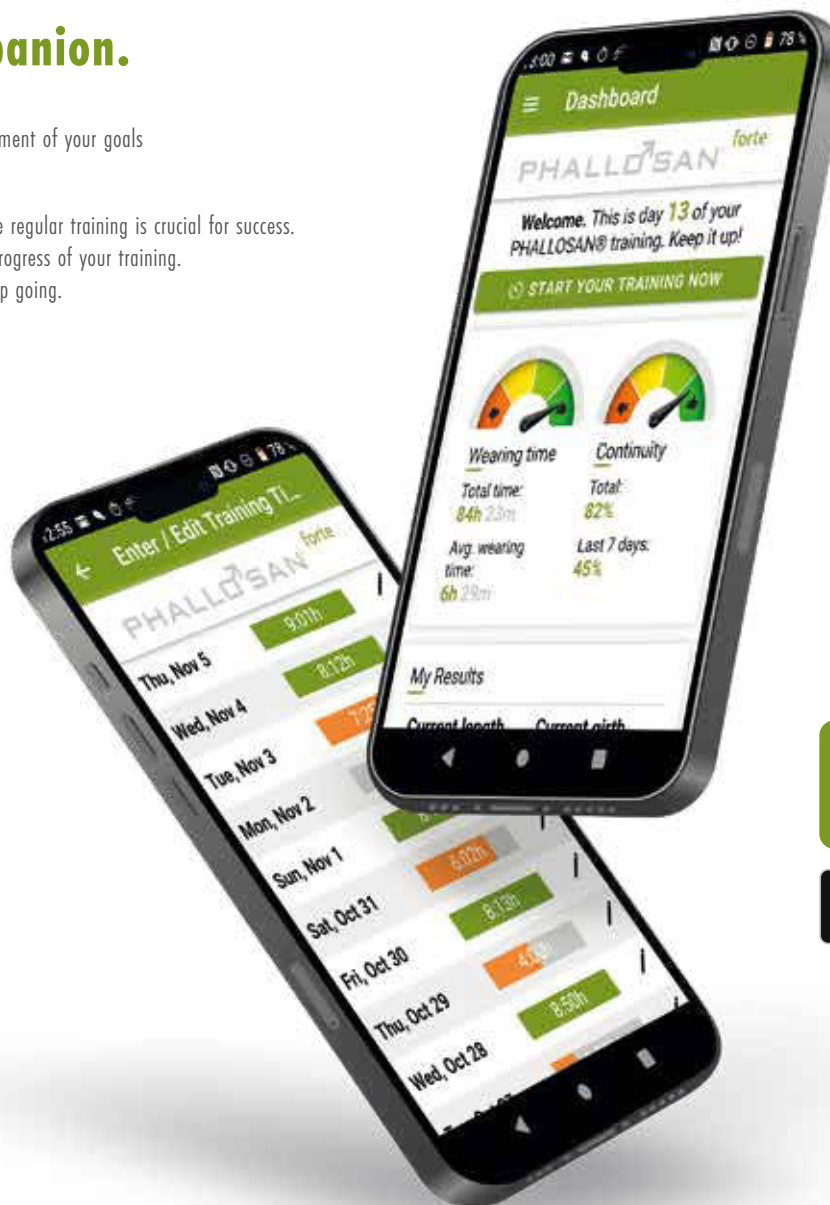
Scan the QR code with your smartphone or tablet to download the app:

iOS

Android

Discover your effective and
motivating personal trainer!

FOLLOW US ON:

Instagram — phallosan_original

Facebook — Phallosan

PHALLOSAN® forte
 PHALLOSAN® plus+
 PHALLOSAN® vertical

READ THE COMPLETE INSTRUCTION FOR USE BEFORE USING THE PRODUCT!

WARNING: THE FOLLOWING PRECAUTIONS MUST BE OBSERVED

The device **must not** be worn during bowel movements, during sports activities, during intercourse or other strenuous physical activity. Neither should it be worn after consumption of alcohol or ingestion of pain-killers or euphorants or if there are

- any wounds or infected areas on the penis or where PHALLOSAN® is attached at the body.
- If pain, numbness or discoloration of the penis head,
- or any other untoward reaction occurs the device must be loosened and removed immediately.

In case of wounds on the penis the device **must not** be worn until these have healed entirely.

In the case of surgery, **do not** use PHALLOSAN® forte/PHALLOSAN® plus+ until your doctor tells you to.

Do not use PHALLOSAN® forte/PHALLOSAN® plus+ if you have problems with venous problems or connective tissue weaknesses.

Do not use the device when you have any kind of piercing at the genital area.

The device **should not** be worn while moving in areas where there is an

increased risk of falling and caution must be exercised when climbing stairs, walking on slippery floors etc.

The device **should not** be worn neither when handling any kind of machinery, motorized vehicles or hot objects nor when bycicle or cooking.

Do not use any creams, Vaseline and other moisture substances while wearing/using the device.

Use the device **only as we describe** in the manual, **do not** use any other materials then the ones we provide you or use other wearing techniques then the ones we tell you in the manual.

We **do not** take any responsibility and we are not liable for any damage caused through use of products or services purchased through this website, be it indirect, special, incidental or consequential damages (including but not limited to damages for loss of business, loss of profits, interruption or the like). If you have any questions regarding the terms of use outlined here, please do not hesitate to contact us at any time.

Swiss Sana AG
 Kirchstrasse 3
 9490 Vaduz
 FL-Liechtenstein

00423 235 84 77

Email: service@swissana.com

In case of risks or side effects
 please contact your doctor
 or our PHALLOSAN hotline

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